



The Scar Heal Kit is designed to help patients prevent and manage post-procedural scarring. Each of combination of sheeting and gel targets a specific set of surgeries, including five specific to breast procedures.



**DAY/NIGHT SYSTEM COMBINING TWO PROVEN SILICONE PRODUCTS**



**13 DIFFERENT COMBINATIONS TARGETING SPECIFIC SURGERIES**



**SIMPLE APPLICATION WITH NO ASSISTANCE FROM PHYSICIAN**



**IMPROVED SCAR RESOLUTION HIGHLIGHTS YOUR WORK**

Item #	Product Name
21503	Scar Heal Kit - 1.5" x 3"
21505	Scar Heal Kit - 1.5" x 5"
21509	Scar Heal Kit - 1.5" x 9"
20112	Scar Heal Kit - 1" x 12"
20122	Scar Heal Kit - 1.5" x 3"
20305	Scar Heal Kit - 3" x 5"
20408	Scar Heal Kit - 4" x 8"
21909	Scar Heal Kit - C-section Procedures

Item #	Product Name
20000	Scar Heal Kit - 3" Breast Circle
21234	Scar Heal Kit - Breast Piece
23456	Scar Heal Kit - Breast Lollipop
25678	Scar Heal Kit - Breast Anchor
21903	Scar Heal Kit - 1.5x3" Breast

## SCAR MANAGEMENT SYSTEM



### Scar Fx AT NIGHT

At night, place Scar Fx over your clean and dry scar for 8-12 hours. The occlusive barrier provided by the silicone sheet helps create the ideal healing environment for your scar.



### RejuvaSil DURING THE DAY

During the day, apply RejuvaSil 2-3 times to help maintain the healing environment. RejuvaSil also helps relieve itchiness and discomfort associated with scars.



### Scar Esthetique FOR DISCOLORATION

Once your scar is flat and smooth, apply Scar Esthetique 3-4 times daily to target remaining discoloration. Many patients see continued results through long-term use.

## FAQ'S

### Why use the Scar Heal Kit?

The Scar Heal Kit provides a standard of care for patients concerned with scarring. Silicone is the gold standard in scar management. The Kit combines silicone products to maximize your patients' results.

### When should you use it?

Patients can begin using the kit as soon as their incision is fully closed. Prompt application helps prevent problem scarring. The Kit can be applied at home with no physician assistance.

### How does it compare to other options?

Scar management can be complicated. The Scar Heal Kit keeps it simple. Alternating sheeting and gel maintains the ideal healing environment and supports skin health. The system allows the skin to breathe and relieves itchiness and discomfort.

# SCAR HEAL KIT

## Effective Silicone Combination


The Scar Heal Kit includes Scar Fx® Silicone Sheeting, RejuvaSil® Silicone Scar Gel, and a roll of hypo-allergenic medical tape. This gives your patient everything needed to effectively prevent or manage their post-procedural scarring.

The patient to the right completed our **Scar Management System** following her **Mohs Procedure**. Improvements from Photo 1 to Photo 2 were accomplished with the Scar Heal® Kit. The final photo was taken after using Scar Esthetique® for discoloration.



DAY 0

DAY 30  
After Scar Fx and RejuvaSil

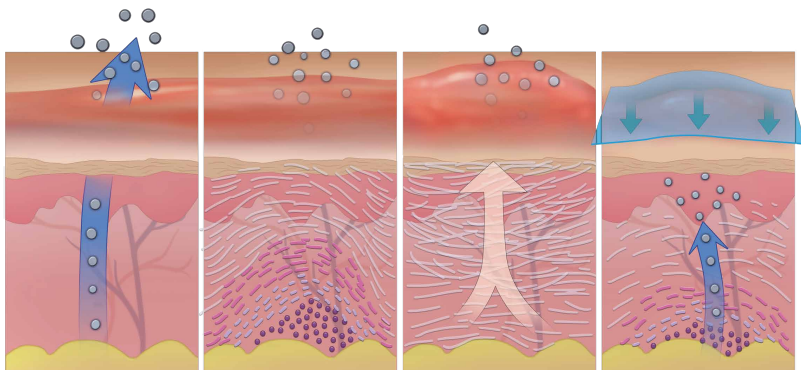DAY 60  
After Scar Esthetique


## How Silicone Works

Silicone creates an occlusive barrier that enhances the body's natural healing process. This occlusion reduces excessive water loss and helps limit excessive collagen production.

*The International Advisory Panel on Scar Management supports Silicone Gel Sheeting as the primary treatment for surgical and traumatic scars. Silicone is the gold standard in topical scar management.*

## TESTIMONIALS

★★★★★

SALLY B.

I used the Kit to improve the appearance of the scar caused by knee replacement surgery. I enjoyed seeing the steady progress as the scar reduced in size and color. I had confirmation of the good results from a massage therapist who remarked on the flatness of the scar. When I replace my other knee, I will be sure to use the Scar Heal kit again.

★★★★★

STEPHANIE H.

Scar Fx and RejuvaSil really worked for my c-section scar! I used cocoa butter for about a year, but saw minimal change. Then I started using the silicone sheeting and silicone gel by Rejuvaskin! These two products helped flatten and smooth the scar. Now, I'm using Scar Esthetique Scar Cream to finish the job!

★★★★★

DR. DAVID HALPERN, MD

We have strongly recommended Scar Heal products for our plastic surgery patients for the past 5 years. With optimal surgical techniques along with the Scar Heal products we are able to obtain the best results possible. From small children to adults they are user friendly and are an affordable, reliable and safe treatment.

Art #SHKSHEET211



A non-invasive, non-surgical solution to minimize surgical scars, traumatic scars, keloids, hypertrophic scars, and burns. The silicone scar sheets are 100% healthcare grade silicone that is self-adhering and reusable for 30-45 days.



100% HEALTHCARE GRADE SILICONE, SELF-ADHERING AND EASY TO USE



13 DIFFERENT COMBINATIONS TARGETING SPECIFIC SURGERIES



SOFTEN, FLATTEN, AND PREVENT KELOID AND HYPERTROPHIC SCARS



RECOMMENDED BY PHYSICIANS AND DERMATOLOGISTS

Item #	Size - Commonly Used
24503	1.5x3" (3.75x7.5cm) - Mohs Surgery, Trocar
24505	1.5x5" (3.75x12.5cm) - Chest, Orthopedic
24509	1.5x9" (3.75x22.5cm) - C-Section, Orthopedic
24112	1x12" (2.5x30cm) - Arm Lift, Tummy Tuck
24122	1x22" (2.5x55cm) - Tummy Tuck, Thigh Lift, Bariatric
24305	3x5" (7.5x12.5cm) - Burns, Traumatic Scars, Face Lift
24408	4x8" (10x20cm) - Burns, Traumatic Scars

Item #	Size (Includes 2 Sheets)
24000	3" Breast Circle
24234	Breast Piece
24456	Breast Lollipop
24678	Breast Anchor
24903	1.5x3" Breast

## Scar Fx Testimonials

★★★★★

Dani, *Patient Review*

This product is amazing. It works really fast. The long term results are going to be better than I expected. I have sensitive skin and can not wear it daily because the medical tape to hold it in place aggravates my skin. I completely recommend this product.

★★★★★

Joyce, *Patient Review*

I am amazed at how quickly I have seen incredible results in the scars from my recent surgery 4 weeks ago. Literally 4 days of use and my scars have almost vanished. I wish I had taken photos because it's so hard to believe the transformation and if I hadn't seen it with my own eyes I would have been so skeptical.

★★★★★

Dr. David Halpern, MD

We have strongly recommended Scar Heal products for our plastic surgery patients for the past 5 years. With optimal surgical techniques along with the Scar Heal products we are able to obtain the best results possible. From small children to adults they are user friendly and are an affordable, reliable and safe treatment.

## FAQ'S

### Does silicone sheeting work?

According to the International Advisory Panel on Scar Management silicone gel sheeting is the gold standard in scar management. Scar Fx is recommended by plastic surgeons and dermatologists around the world.

### When should you use it?

Patients should use silicone sheeting as soon as their incision is fully closed. Prompt application helps prevent problem scarring. Scar Fx can be applied at home with no physician assistance.

### How does it compare to other options?

Scar Fx is a completely non-invasive solution to any scar. Each sheet is made of 100% healthcare grade silicone. Scar Fx has a simple application. Our silicone sheets are very durable and can be washed and reused for around 30-45 days.




## Why Use Silicone Sheeting

Scar Fx Silicone Sheeting is a non-surgical, non-invasive solution to soften and smooth problem scarring. It is completely painless and low-cost. Scar Fx improves the color and works to enhance the body's own natural healing process to soften and flatten raised scars.

Our silicone sheets are made of 100% healthcare grade, self-adhering silicone that is soft, pliable, and contours easily to the body. Each sheet can be washed and reused for around 30-45 days. View the sizing chart below to see all of our silicone sheeting shapes.



## Scar Fx® Size Chart

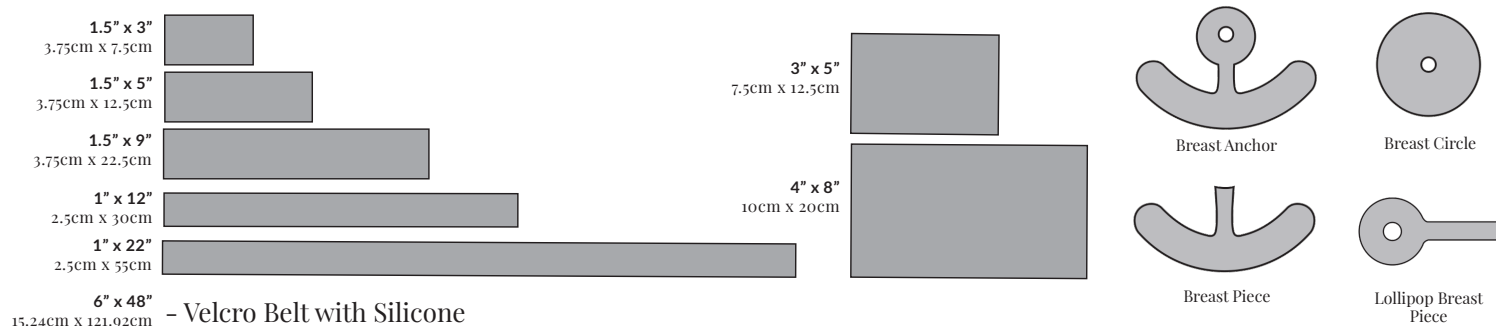

## Scar Fx® Silicone Sheeting SECC

**Product Description:** Scar Fx Silicone Sheeting is made of 100% healthcare grade silicone, manufactured via a proprietary method allowing the sheeting to be flexible, durable, and have a natural tack without the addition of adhesives.

**Product Indications:** Scar Fx Silicone Sheeting is used to improve the appearance of surgical and traumatic scars (including normal post-op scars, hypertrophic scars, and keloid scars) and burns. Scar Fx is applied to a clean and dry scar, for up to 8-12 hours nightly, and can be secured to the skin with medical tape as needed. Scar Fx creates an occlusive barrier promoting skin moisture. This barrier supports the mechanism of action, which includes hydration and occlusion.

### SAFETY

Scar Fx Silicone Sheeting is classified by the FDA/CE as an OTC Class 1 Medical Device, which is 510K/PMA-Exempt.

Scar Fx is indicated for use on epithelialized (closed) skin. Scar Fx has been in the market since 1998, is sold worldwide, and has an excellent record of safety with no adverse events reported. Silicone is naturally hypoallergenic, making this product safe for use on all skin types. Scar Fx is not recommended for patients under the age of three due to the risk of choking hazard. Please consult your physician.

### EFFICACY

The International Advisory Panel on Scar Management recommends silicone sheeting as the "... standard care for plastic surgeons."<sup>(1)</sup> for non-invasive scar management.

Silicone Sheeting has been studied in excess of thirty years, and is considered the first line of defense against abnormal scarring. Studies conclude that "It is much more efficient to prevent hypertrophic scars than to treat them."<sup>(1)</sup>

Scar Fx Silicone Sheeting is made with the highest quality ingredients and to the highest specifications to achieve the maximum result for patients seeking to manage new and old scars.

### COST

Scar Fx provides a tremendous value to both the physician and patient based on its quality, durability, and ease of use. A single Scar Fx silicone sheet can be used for the duration of treatment (30-45 days for most normal surgical scars).

Physicians receive preferential pricing due to the positioning of the product as physician-recommended. Scar Fx Silicone Sheeting has an industry-low return rate under 0.05% compared to a 2% industry average.

### COMPLIANCE

Scar Fx Silicone Sheeting's application time was designed to maximize compliance. Scar Fx is applied in the evening and worn while sleeping. In the morning, the sheeting is removed and the process is repeated.

(1) Mustoe TA, Cooter RD, Gold MH, Hobbs FD, Ramelet AA, Shakespeare PG, Stella M, Téot L, Wood FM, Ziegler UE; International Advisory Panel on Scar Management. Plast Reconstr Surg. 2002 Aug;110(2):560-71.





A soft, gentle, non-sticky silicone gel designed to soften and flatten raised scars and help relieve itchiness and discomfort associated with scar healing. It's completely discreet and combines silicone with additional ingredients effective for scar management.



97% LIQUID SILICONE PROVEN TO SOFTEN AND FLATTEN SCARS



CLEAR AND ODORLESS GEL FOR DISCREET SCAR MANAGEMENT



RELIEVE ITCHINESS AND DISCOMFORT ASSOCIATED WITH SCAR HEALING



GENTLE FORMULA SUITABLE FOR FACIAL AND PEDIATRIC SCARS

Item #	Size - Commonly Used
80000	30mL/1 Fl. Oz. RejûvaSil® Silicone Scar Gel
80015	15mL/0.5 Fl. Oz. RejûvaSil® Silicone Scar Gel
80010	10mL/0.338 Fl. Oz. RejûvaSil® Silicone Scar Gel

## RejûvaSil Testimonials



### Paula, Patient Review

I've tried a couple of other over the counter scar gels and creams. RejûvaSil hides and conceals the scar much better and a little goes a long way! Highly recommend this gel!



### Clare, Patient Review

Purchased this for my mother after she underwent extensive surgery for skin cancer on her face. She liked the product immediately because of the way it seemed to "melt into the scar" and she remarked that it lessened the pain and tightness she was experiencing.



### Rosie, Patient Review

I have already seen an amazing improvement with using for a week. My scars feel softer and look lighter. I am very pleased with the results so far and cannot wait to see what it is like after a month!

## FAQ'S

### How does the silicone gel work?

RejûvaSil leaves a clear layer of silicone on top of the scar to reduce collagen production and soften and flatten raised scars. Additional ingredients help soothe the skin and relieve itchiness and discomfort.

### When should you use it?

Patients should use RejûvaSil as soon as their incision is fully closed. It is excellent for facial and pediatric scars. We also recommend using the gel alongside silicone sheeting after surgery.

### How does it compare to other gels?

RejûvaSil is 97% liquid silicone with additional ingredients to improve texture and performance. Unlike other gels, RejûvaSil feels silky smooth on the skin and is not oily. It also dries quickly and is completely odorless.





## Discreet solution for surgical and traumatic scars.

RejûvaSil combines silicone with Vitamin C, Squalane, and Emu Oil. This uniquely crafted formula helps prevent significant scarring and helps soothe the skin during the healing process.



## RejûvaSil® Silicone Scar Gel SECC

**Product Description:** RejûvaSil Silicone Scar Gel is a topical liquid silicone gel, made with a proprietary formula designed to dry quickly and leave a thin, transparent film that does not flake off on the skin.

**Product Indications:** RejûvaSil Silicone Scar Gel is used to improve the appearance of surgical and traumatic scars (including normal post-op scars, hypertrophic scars, and keloid scars) and burns. It is applied to a clean and dry scar up to 3 times daily. RejûvaSil dries to a thin semi-occlusive film on the skin promoting a moisture barrier. This barrier supports the mechanism of action which includes hydration and occlusion.

### SAFETY

RejûvaSil Silicone Scar Gel is classified by the FDA/CE as an OTC Class 1 Medical Device, which is 510K/PMA-Exempt.

RejûvaSil is indicated for use on epithelialized (closed) skin. RejûvaSil has been in the market since 2009, is sold worldwide, and has an excellent record of safety with no adverse events reported.

Silicone is naturally hypoallergenic, making this product safe for use on all skin types, including younger patients. Please consult your physician.

### EFFICACY

Silicone gels are widely accepted alternatives to silicone gel sheeting and are considered a "...first choice for preventing hypertrophy of recent scars." [1]

An independent research laboratory conducted a monadic 8 week pilot case series evaluation specifically on RejûvaSil Silicone Scar Gel. Expert clinical grading concluded that all participants exhibited improvements in texture/smoothness, as well as improvement in overall appearance. Seventy percent of the study participants exhibited a decrease in scar length measurement. Clarity Lite photography showed a directional improvement in lightening of the scar after 4 weeks. Maximum improvement was noted at 8 weeks. The p value was noted to be <0.001.

### COST

RejûvaSil has an industry-low cost profile. Products of comparable quality retail approximately 2 to 2.5x the cost of RejûvaSil and often have a lower silicone content.

Physicians receive preferential pricing due to the positioning of the product as physician-recommended and an industry-low return rate.

### COMPLIANCE

RejûvaSil's proprietary formula and blend of silicones was specifically designed for patient compliance. RejûvaSil is easy to apply during the day and while on the go.

RejûvaSil dries quickly, is odorless, non-greasy, non-tacky, and has a pleasant satin-like texture. RejûvaSil is discreet, often unnoticeable, and both sunscreen and makeup can be applied once dried. RejûvaSil's attributes maximize patient compliance and therefore efficacy.

(1) Signorini, M. & Clementoni, M.T. Aesth Plast Surg (2007) 31: 183. doi:10.1007/s00266-005-0122-0



Item #	Size - Commonly Used
60000	60mL/2 Fl. Oz Scar Esthetique® Scar Crème
60030	30mL/1 Fl. Oz Scar Esthetique® Scar Crème
60010	10mL/0.338 Fl. Oz. Scar Esthetique® Scar Crème

An ingredient-packed silicone scar cream designed to soften and brighten your scar. Scar Esthetique targets discoloration at the cellular level and supports overall skin health using a unique blend of botanicals, polypeptides, anti-oxidants, and silicone.



23 EFFECTIVE INGREDIENTS INCLUDING SILICONE



TARGETS DISCOLORATION AT THE CELLULAR LEVEL TO FADE SCARS



KEY BOTANICALS, POLYPEPTIDES AND ANTI-OXIDANTS



RECOMMENDED FOR OLD SCARS AND LONG-TERM SCAR CARE

## Scar Esthetique Testimonials



Michelle, *Patient Review*

Used on scars from current and past foot surgeries and plastic surgery. The results are way better than the suggested use of Vitamin E or Cocoa Butter. I also see results a hand scar that is over 20 years old. This is the best and highly recommended.



Susan, *Patient Review*

I can't believe how quickly this stuff works. I have scars from a bleach spill, and from a skin disorder, and the Scar Cream with Silicone is quickly fading them all. I had no idea such a product existed. Thanks, Rejuvaskin! :)



Vicki, *Patient Review*

This cream seems to have helped old mosquito bite scars and other scars on my daughter's ankles. She extremely sensitive to mosquitoes and so ends up scratching them which causes scarring. She has used other creams but this one seems to have better results.

## FAQ'S

### How does the Scar Esthetique work?

Scar Esthetique is a silicone-first scar cream combining anti-oxidants, polypeptides, and botanicals to improve scars. The cream targets discoloration at the cellular level to soften and brighten scars.

### When should you use it?

We recommend Scar Esthetique for discoloration and old scars. It's also used as part of our Scar Heal System to address any remaining discoloration and long term care after using the sheeting and gel.

### How does it compare to other creams?

Scar Esthetique combines 23 ingredients effective for scar management. These ingredients work to nourish the skin and address discoloration. Scar Esthetique has a high compliance rate because it is non-greasy and easily absorbed.



## Key ingredients in addition to silicone.



**Onion Bulb Extract** is a valuable source of Vitamin A, Vitamin B, and Vitamin C and a protein with anti-bacterial and anti-inflammatory properties.



**Calendula Flower**, or Pot Marigold, has antibacterial, anti-inflammatory, and anti-oxidant properties.



**Polypeptides** help stimulate cell communication and repair related skin damage while limiting levels of inflammatory proteins and loss of healing potential.



**Arnica Montana Flower** has anti-inflammatory properties and add suppleness to the skin and also may help reduce inflammation and bruising.



## Scar Esthetique® Scar Crème with Silicone SECC

**Product Description:** Scar Esthetique Scar Crème with Silicone is a topical cream made with Silicone and 22 additional ingredients including Anti-Oxidants, Polypeptides, and Onion Bulb Extract to penetrate deep into the epidermal layers, to nourish and moisturize. Scar Esthetique improves scar cosmesis by focusing on normalizing scar discoloration.

**Product Indications:** Scar Esthetique Scar Crème with Silicone is used to improve the appearance of surgical, traumatic scars (including normal post-op scars, hypertrophic scars, and keloid scars) and burns. Scar Esthetique is gently massaged into a clean and dry scar 3 to 4 times daily.

### SAFETY

Scar Esthetique is indicated for use on epithelialized (closed) skin. Scar Esthetique has been in the market since 1999, is sold worldwide, and has an excellent record of safety with no adverse events reported. Scar Esthetique is not recommended for children under the age of 12. Please consult your physician.

Scar Esthetique Scar Crème with Silicone is classified as a cosmetic.

### EFFICACY

A monadic 8 week pilot case series evaluation of Scar Esthetique for scar management to safely improve scar appearance of scars was completed with the following outcomes:

- 100% of study participants exhibited improved texture/smoothness of the scar.
- 90.9% of the study participants exhibited overall improvement in the appearance.
- 50% of subjects exhibited a decrease in scar length measurement.

Image analysis using Clarity Lite showed a statistically significant improvement in the texture of the scar after 4 weeks of use.

### COST

Scar Esthetique has an industry-low cost profile. Products of comparable quality retail approximately 4x the cost by volume.

Physicians receive preferential pricing due to the positioning of the product as physician-recommended. Scar Esthetique has an industry-low return rate.

### COMPLIANCE

Scar Esthetique's proprietary formula and blend of ingredients were specifically designed for patient compliance. Scar Esthetique is easy to apply during the day and while on the go.

Scar Esthetique absorbs quickly, is discreet, often unnoticeable, not greasy and both sunscreen and makeup can be applied shortly after application. Scar Esthetique's attributes maximize patient compliance and therefore efficacy.

SSCSHEET211







## Skin Recovery Cream

A radiation relief cream that helps fortify, soothe, and revitalize patients' skin before, during, and after radiation treatment.

### COLLAGEN PRECURSOR BLEND

A proprietary blend of ingredients confirmed to increase pro-collagen, elastin, and hyaluronic acid synthesis.

### ALOE VERA

Increased the median time prior to any skin reaction during radiation to five weeks, two additional weeks.<sup>2</sup>

### HYALURONIC ACID

Reduced incidence of high-grade radio-epithelitis and improved quality of life for radiation patients.<sup>3</sup>

### CALENDULA

Proven to be more effective than traditional methods at preventing acute dermatitis grade 2 or higher.<sup>1</sup>



<sup>1</sup> Pommier P, et al. J Clinical Oncol. 2004; 22:1447-1453

<sup>2</sup> Olsen DL, et al. Oncol Nurs Forum. 2001; 28:543-247

<sup>3</sup> Liguori V, et al. Radioth Oncol. 1997; 2:155-161

## TESTIMONIALS



### Laura, Nurse Review

I had a patient who had a lumpectomy and received 26 radiation treatments. That patient just finished treatment and had zero redness with the skin recovery cream. I really loved it and the texture is amazing. We have another moisturizer that drags across the skin, this just simply glides!



### Stacy, Patient Review

I started the skin recovery cream on the first day I started my treatments. I'm now starting my fifth week, and I'm having no issues with my skin hurting. I'm a little pink from the radiation, but I've been putting it on 4 times a day. It's so smooth and easy to apply. I love the smell of it. I would recommend this product to anyone who is having radiation treatments. It's very soothing on the skin.



### Karen, Patient Review

This cream is amazing! I applied it 2-3 times a day throughout my radiation treatment for breast cancer. It was soothing, healing and softening to my radiated skin. I continued to use the product post-radiation and plan to use it indefinitely. My physicians raved about how well my skin looked at all appointments. My surgery scars are barely visible now too!!

## FAQ'S

### Why use our Skin Recovery Cream?

Rejûvaskin's Skin Recovery Cream is a super-moisturizer that keeps skin hydrated and better equipped to handle stress.

Maintaining skin health during radiation increases patients' comfort and improves their ability to complete scheduled treatments.

### When to use our Skin Recovery Cream?

Use before scheduled treatments to fortify skin prior to radiation damage.

Use after individual radiation treatments to soothe and support patients' skin.

### How does this compare to other treatments?

Our Skin Recovery Cream uses botanicals proven to reduce inflammation, rather than harsh steroids, making it safe for continual use on all skin types.


It's more than just soothing. We combine evidence-based ingredients that fortify skin by increasing pro-collagen and elastin production.





## PROVEN EFFECTS ON ESSENTIAL SKIN COMPONENTS

Our unique formula not only soothes the skin, but improves its ability to supply new, healthy skin cells by increasing the synthesis of key structural components.

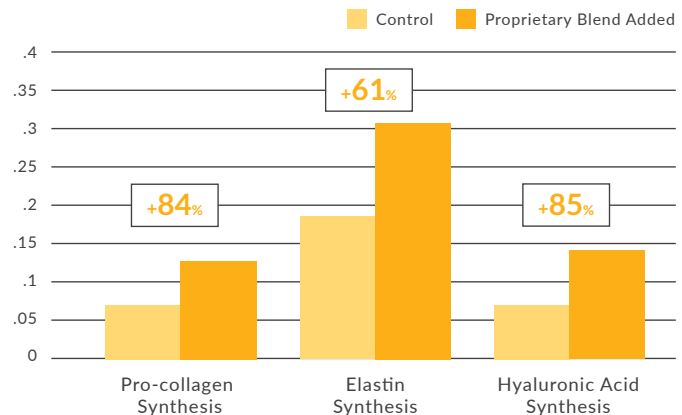

## WHY THESE COMPONENTS ARE IMPORTANT

### ELASTIN SYNTHESIS

Elastin gives the skin its ability to stretch and contract. It works with collagen to form connective tissue in the skin and is needed for the formation of dermal layers.

### COLLAGEN PRODUCTION

Collagen is the main structural protein found in the skin. Collagen is essential for growth and repair of skin affected by radiation therapy.

### HYALURONIC ACID SYNTHESIS

Hyaluronic Acid retains moisture at an incredible rate. Well-hydrated skin prevents dryness, supports the skin's protective barrier, improves elasticity, and aids in regeneration.

### PROLIFERATION AND DIFFERENTIATION

Cell differentiation and proliferation are the methods the body uses to supply the new, healthy cells necessary for maintaining a functional skin barrier.



## HOW THIS MAKES US DIFFERENT

We use a proprietary blend of bamboo, pea, and glucosamine to support skin's essential structural components. Our Skin Recovery Cream is more than just soothing, it prepares the skin for radiation and supports skin after treatment.

Art #SRC SHEET211

A stretch mark cream developed with new moms in mind that helps minimize the likelihood of developing stretch marks and reduce the appearance of those that have already occurred.



## PALMITOYL HEXAPEPTIDE 12

Palmitoyl Hexapeptide-12 creates a response in the skin that stimulates collagen and elastin fibroblasts.

## COCOA BUTTER

Prevents dryness and is a rich source of anti-oxidants. It also soothes and softens the skin.

## CERAMIDE-2

A waxy lipid that helps hold moisture and prevent the breakdown of collagen.

## ALOE VERA

Aloe Vera soothes and hydrates the skin and has qualities of an anti-oxidant.

## Product Testimonials



### Nicole, User Review

I have bought this and was a little skeptical because I've tried many creams but after using this a couple days it has reduced my stretch marks a lot! Hopefully they will completely be removed by the time I finish this bottle. I will definitely be purchasing more of this if needed.



### Savaya, User Review

I used this cream daily starting in my second trimester and now I'm 3 months postpartum with minimal stretch marks! The couple of stretch marks that I have improved greatly while using the cream! Would definitely recommend.



### Nancy, User Review

During mid-pregnancy I noticed that I was developing stretch marks on my lower stomach. A friend of mine suggested I try this cream and I am so thankful that she did! I literally noticed my skin becoming firmer and the stretch marks fading after less than a week of using it. The stretch marks are almost gone!

## The Need for the Stretch Mark Cream

Your skin is going to be with you a long time. It's a big part of how people see you. We encourage you to embrace your skin health and be confident in it. However, if stretch marks are causing you to lose confidence, the Stretch Mark Cream could be your solution! It's more than erasing stretch marks. It's about improving how you feel about your skin, taking back your confidence, and embracing the journey that brought you here.





## MORE THAN MOISTURIZING

Rejûvaskin® Stretch Mark Cream combines proven botanicals with polypeptides that help stimulate collagen production and ceramides that prevent collagen breakdown.



## OBGYN-RECOMMENDED

It's recommended by physicians and sold in pharmacies all over the world to help reduce the likelihood and appearance of stretch marks. The Stretch Mark Cream is safe for all skin types.

# MORE THAN A MOISTURIZER



## SAFE FOR NEW MOMS

The formula was designed to help new moms prepare their skin for upcoming growth. The fragrance-free and paraben-free formula is safe to use after the first trimester.



## ANY STRETCH MARKS

Don't let the pink fool you, it's not only for pregnancy. It's great for growing teens, weight lifters, or anyone experiencing rapid skin changes resulting in stretch marks.



BEFORE



AFTER

## BEFORE AND AFTER

We know that it's important to see the difference. Each of our before and after photos are user submitted and feature real patients with real results. This new mother used Rejûvaskin® Stretch Mark Cream after delivery to target her stretch marks.

## FAQ'S

### What causes stretch marks?

Stretch marks, or striae, are caused by the stretching of the skin during rapid growth. They are often associated with pregnancy but can also be a result of weight training, growth, or weight change.

### Is it only for stretch marks from pregnancy?

No! Our Stretch Mark Cream is great for reducing the appearance of stretch marks no matter the cause. The key is in the formula which was developed to target all types of stretch marks.

### When to use the Stretch Mark Cream?

The Stretch Mark Cream can be used both in advance, to prepare the skin for expected weight change, and after stretch marks occur to improve their appearance. For the best results, use it two times daily.

### Is it safe to use during pregnancy?

Our formula is paraben-free and safe to use after your first trimester. Beginning your stretch mark therapy before the skin is stretched helps condition the skin and reduces the risk of stretch marks.

SMCINFO211

Rejûvaskin®

## Anti-Pollution Facial Cleanser

Pollution can really be tough on your skin, causing issues such as dullness, uneven tone, premature aging, dehydration, irritation, and acne. Protect your skin from damaging particulate matter and free radicals with Rejûvaskin® Anti-Pollution Facial Cleanser.



**GENTLE CLEANSING FORMULA DESIGNED TO REMOVE PARTICULATE MATTER.**



**FIGHTS THE ACTIVITY OF FREE RADICALS TO PREVENT PREMATURE SKIN AGING**



**NATURE INSPIRED MECHANISM PROVEN TO FIGHT POLLUTION-RELATED DAMAGES**



**ENHANCED WITH LEMONGRASS ESSENTIAL OIL FOR A FRESH SCENT AND ANTI-OXIDANTS**

ITEM #	PRODUCT NAME	SIZE
30060	Rejûvaskin® Anti-Pollution Facial Cleanser	60mL/2 Fl. Oz
30010	Rejûvaskin® Anti-Pollution Facial Cleanser	10mL/.338 Fl. Oz

## TESTIMONIALS

COURTNEY H.

This cleanser is my new must have! I live in the Atlanta area and the combo of poor air quality and Summer heat had me breaking out, but my skin looks great now.

SARAH P.

I love this cleanser! It has helped clear up my skin when other products were unable to do the job. It leaves my skin feeling clean and fresh. I would highly recommend this to anyone looking for a new daily cleanser.

MORGAN R.

OBSESSED with the smell! I have super sensitive skin and this cleanser has been so gentle – no dryness or irritation. So glad I found it!

## THE NEED FOR ANTI-POLLUTION PRODUCTS

Air pollution can be very harmful to your skin. Especially when you consider the continued increase in pollution levels across the globe. The problem is particulate matter with a diameter less than 2.5µm (PM2.5) influences surface barrier functions causing harmful effects to your skin. Issues like dullness, uneven skin tone, premature aging, irritation, dehydration, and acne. Our uniquely designed formula helps remove these damaging particles and restore luminous, healthy skin.



rejuvaskin.com | 727-535-0022





# THE SCIENCE OF ANTI-POLLUTION

## THE CLEANSING ACTION

With our key ingredient (EXO-P™), the Anti-Pollution Cleanser forms a mesh able to entrap tiny, damaging particulate matter and remove it from the surface of the skin to prevent and reduce pollution-induced damage.

A study has shown a 27% increase in the percentage of all particulate matter removed when cleansing with EXO-P™ vs. a placebo cleansing with soap and water.

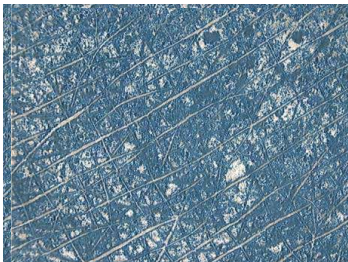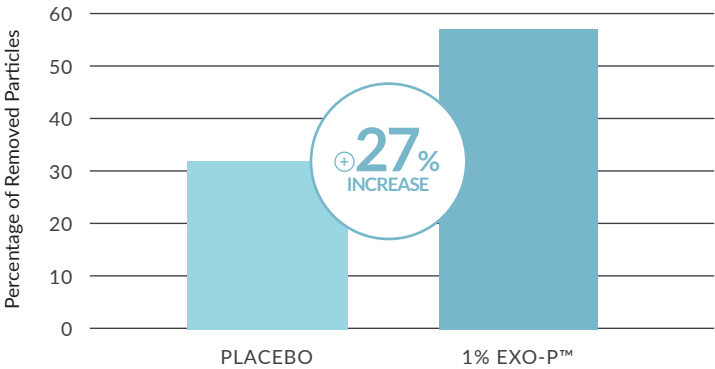

BEFORE CLEANSING



AFTER CLEANSING

**It's more than pollution protection.** This cleanser is also effective in removing dust, dirt, grime, makeup and almost anything else that builds up on the skin.

The study also reports up to an 82% cleansing action when untreated skin is cleansed with a 1% EXO-P™ solution. When less particulate matter is present after cleansing the result is cleaner, healthier skin.

## KEY INGREDIENTS



### ALTEROMONAS FERMENT EXTRACT

EXO-P™ is an exopolysaccharide sustainably sourced from French Polynesia. It forms a mesh able to entrap and remove particulate matter from your skin.



### SODIUM C14-16 OLEFIN SULFONATE

A cleansing ingredient derived from coconuts that removes build-up that occurs throughout the day. It also helps give the cleanser its foamy texture.



### CYMBOPOGON FLEXUOSUS OIL

This lemongrass essential oil gives the cleanser a citrus-like scent. It also has anti-oxidant, anti-fungal, anti-bacterial, and anti-inflammatory properties.




### ALOE VERA

A common skincare ingredient that helps soothe and hydrate the skin. It offers anti-inflammatory, anti-microbial, and anti-oxidant qualities for the skin.





A rich moisturizing cream loaded with skin-restoring ingredients that improve skin elasticity, tone, and firmness.

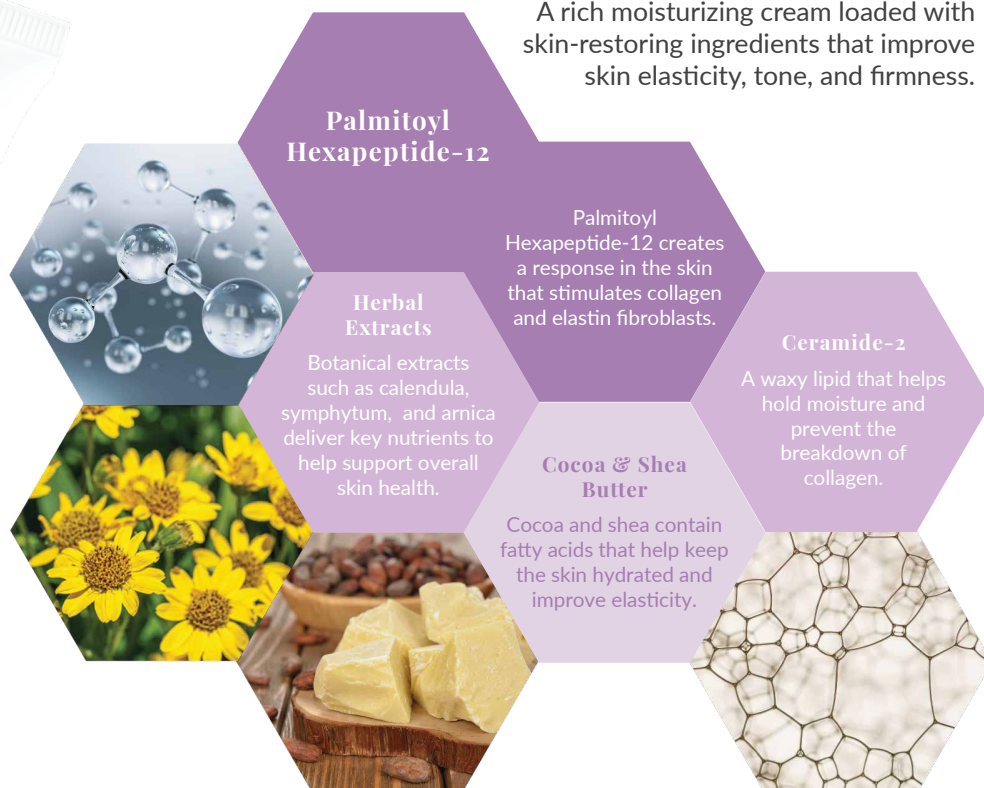

## Product Testimonials

★★★★★

*RM, User Review*

I had trouble ordering my usual Neck and Jaw cream, so when this came out, I tried it. It is the best cream I have ever used on my neck. I no longer order or use the other cream...this is the best. The wrinkles melt away. It is amazing.

★★★★★

*Dada, User Review*

I think this stuff is like a miracle to me! I have used it on my neck which is heavily wrinkled. In a week my neck lines had dissipated and now 2 weeks later, my skin look like it did 20 years ago!! I am 63 years old and wish I had done a before and after photo but my friends can confirm my success!!

★★★★★

*Joni, User Review*

This product is truly a "Pot of Gold." I purchased this cream to help correct the appearance of the fine lines, dry skin on my neck, and décolleté area. This cream is high quality, and you only need a small amount to use when applying. I use it 2 times a day, and it has shown such amazing results in such a short amount of time.

## The Need for the Advanced Firming Cream

It's natural for the skin to lose firmness and elasticity over time. However, external factors like stress, UV exposure, and pollution speed up the process. That's why we created the Advanced Firming Cream.

Effective ingredients support collagen and elastin. These key components help your skin "bounce back" for a natural, firm, and healthy appearance.



## Helps the Skin "Bounce Back"

The Advanced Firming Cream supports collagen and elastin to help the skin bounce back for a natural, firm, healthy appearance.



## Smooths Fine Lines and Wrinkles

Smooth the texture of the skin and reduce the appearance of fine lines, wrinkles, and crepey skin by providing key nutrients and hydration.



## Easily Applied to All Areas

The Advanced Firming Cream is typically suggested for use on the neck, décolleté and body, but can be used anywhere to improve skin texture or tone.



## Unique Blend of Key Ingredients

We use a combination of essential compounds, floral extracts and natural ingredients to ensure only the best results when using the Advanced Firming Cream.



## FAQ'S

### Who should use the Advanced Firming Cream?

As you get older you skin loses the ability to "bounce back." This often causes things like dryness, wrinkles, crepey skin, and even discoloration. The Advanced Firming Cream helps address these issues and restore a healthy, youthful appearance to your skin.

### How often should you use the firming cream?

We recommend applying the firming cream twice daily, once at night and once in the morning. Many people report seeing results overnight, but it's important to be consistent and use it regularly.

### Where can you apply the firming cream?

Anywhere you have a concern! Most people apply it to the neck, décolleté, arms and legs. It's really a great total body cream. Use it wherever you want to improve your skin's appearance and overall health.

### What makes this different than other creams?

Our years of experience and global network have taught us to focus on skin health. We start by using natural ingredients with the expectation of nurturing the skin, so it and you can be and feel healthy.